

## **HERITAGE MEDCALL TECHNICAL SPECIFICATIONS**

## **Model HM-541L LATCHING REMOTE EMERGENCY CALL STATION**

- SINGLE GANG MOUNT
- NYLON PULL CORD
- LED STATUS INDICATOR
- SLIDE SWITCH OR PULL CORD  
EMERGENCY CALL PLACEMENT



### **DESCRIPTION:**

The Sentry Latching Remote Emergency Call Station is single gang, wall mounted plate with a nylon pull cord and LED indicator light. The station provides a latching remote emergency call switch for the Model HM-W541RCS or HM-W541RCL.

### **WIRING:**

The Remote Emergency Call Station is connected via a 6 pin cable HM-6P (included). Up to 6 Model HM-541L remote stations can be "daisy chain" connected to one master station.

### **INDICATOR:**

An LED indicator illuminates when an emergency call has been initiated.

### **RESET:**

Emergency call station reset is accomplished by returning the slide switch to the up position. All emergency calls require this reset action to assure all calls are verified and all initiating locations are visited.

### **PANEL FINISH AND DIMENSIONS:**

The Model HM-541L Remote Emergency Call Station is available in a high-impact, white, molded ABS plastic plate. Maximum plate dimensions are 3.15"W X 4.8"H X 0.4"D overall.

### **INSTALLATION:**

Model HM-541 is installed to a single gang electrical box or plaster ring. The 56" nylon pull cord is normally installed with the free end within 2" off of the finished floor.