



**Installation:**

Remove slide off cover.

Find suitable location and mount unit using suitable hardware. Two mounting holes are provided for #8 screws.

Connect to one or more lighted push-button switches, dome lamps and/or tone units.

Maximum lamp and tone load is 2 Amps.

Route all wires through hole in bottom of enclosure.

Connect to suitable 24 VDC power supply.

Verify Power LED (D5) is on.

If utilizing tone output make sure the enable jumper is in place (default).

While switch input is active adjust the Tone Length (0-1 second) and Tone Rate (repeat 1-60 seconds).

LED D2 shows when the lamp output is active.

LED D3 shows when the tone output is active.

|          |  |   |  |
|----------|--|---|--|
| Title    |  | Heritage MedCall, Inc. Tampa, FL 813-221-1000 |  |
| Size     |  | HM-T1 Lamp and Tone Control Unit              |  |
| Number   |  | A   |  |
| Date     |  | July, 2004                                    |  |
| Filename |  | T1Inst.sch                                    |  |
| Drawn by |  | jt  |  |
| Sheet    |  | 1 of 1  |  |