



**Installation:**  
 Remove two cover screws and remove top cover.  
 Find suitable location and mount unit using proper hardware. Two mounting holes are provided for #10-12 screws.  
 Connect to one or more lighted push-button switches and dome lamps per circuit and one or more tone units as required.  
 Maximum lamp and tone load is 2 Amps.  
 Route all wires through hole in side of enclosure.  
 Connect to suitable 24 VDC power supply.  
 Verify Power LED (D12) is on.  
 If utilizing tone output make sure Tone Enable switch for each channel is in the "ON" position (default).  
 While any switch input is active adjust the Tone Length (0-1 second) and Tone Rate (repeat 1-60 seconds).  
 LED's D1, 3, 5, 6 indicate when the lamp outputs are active.  
 LED D11 shows when the tone output is active.

Title		Heritage MedCall, Inc. Tampa, FL 813-221-1000	
Size		HM-T4 Lamp and Tone Control Unit	
Date August, 2004		Rev	
Filename T4Inst.sch		Drawn by jt	
Sheet 1 of 1		Rev	