


4

3


2

1

**Optional Hook-up for Dome Lamp and/or Secondary Input**

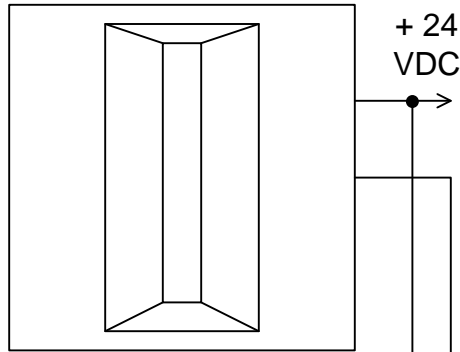

Input 2 (IN2)  
Normally Open Circuits  
(Smoke Detectors)

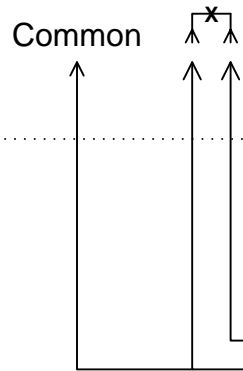

**Installation Notes:**

Insert 3 volt lithium battery noting correct polarity orientation. Log on to the Sentry computer using an administrator password. Choose "Transmitters " option then select "New" then "Enroll". On the backside of the HM-WSPS press the reset switch (SW2) for one second. Serial number and type will be automatically filled in. Edit Announcement if required and press "Save". Press "Room" and select location from list and press "Apply" and then "Save". Press "Resident" and select name and "Apply" and "Save". Refer to Sentry Freedom manual for full programming instructions.

**LED Operation:**




Active: ON  
Reset: OFF

**Note:** Press reset button after changing battery.

*This device has not been tested by UL as a component of a UL 1069 Listed Visual Nurse Call System.*

**Optional Devices:**

Using the supplied HM-4P connector provides hook-up for driving a 24 volt Dome Lamp and input from a secondary input device using a normally open contact closure.

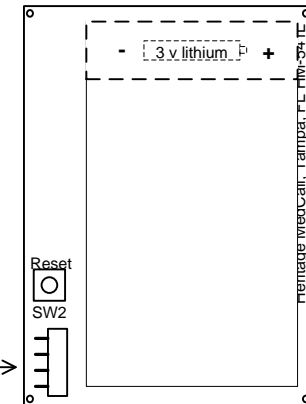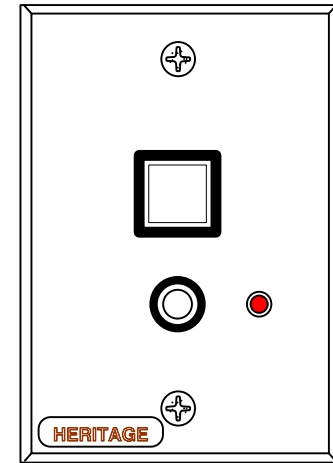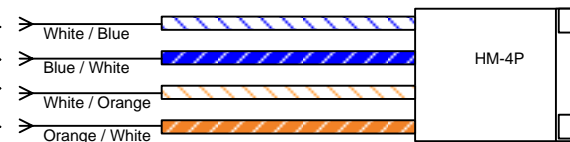

**Installation:**

Model HM-WSPS is installed to a single gang non-metallic electrical box or surface mounted utilizing optional wire-mold box and suitable wall anchors for the wall material. Unit must be mounted using nylon hardware provided. DO NOT over tighten the nylon screws to avoid stripping the threads.

**Specifications:**

Current Draw: Quiescent: 5uA  
Call Placed: 3mA  
During Transmission: 50mA  
Power: 3 Volt Lithium Battery Supplied



202 East Virginia Ave  
Tampa, Florida 33603

813-221-1000 (ph)  
813-223-1405 (fax)

www.heritagemedcall.com

**Sentry Freedom Wireless Emergency Call Systems**

**HM-WSPS**

**Wireless Single Patient Station**

SIZE	Drawn by: JRP	File Name: HM-WSPS insert.pdf	REV B
SCALE	NA	Date: 09-25-09	SHEET 1 OF 1

4

3

2

1

D

C

B

A

D

C

B

A