

HERITAGE MEDCALL TECHNICAL SPECIFICATIONS

Model HM-671 BED CORD WALL-PLATE

- SINGLE GANG MOUNT
- ACCEPT STANDARD 1/4" BED-CORDS
- LED STATUS INDICATOR



DESCRIPTION:

The Sentry Bed Cord Wall-Plate is a single gang, wall mounted plate with a standard 1/4" phone jack and LED indicator light. The station provides a remote bed-cord emergency call switch for the Model 527 Host Panel. Standard momentary bed cords with 1/4" phone plug (such as Sentry Model HM-672, HM-673 or HM-674) are used for call placement. In addition if the bed-cord is pull from the plate a call will also be placed.

WIRING:

The Remote Emergency Call Station is connected to the host panel with a 3 conductor, 22 ga. cable. Up to 6 Model HM-671 and Model HM-541 remote stations can be "daisy chain" connected to one host. Connections are made with prewired pigtailed. The last Remote Station to be wired to a host input must be installed with a EOL Resistor, Model HM-585.

INDICATOR:

An LED indicator illuminates when an emergency call has been initiated and verified by the Sentry Master Console.

RESET:

Emergency call station reset is accomplished by pressing the CHECK-IN button on the host panel. All emergency calls require this reset action to assure all calls are verified and all initiating locations are visited.

PANEL FINISH AND DIMENSIONS:

The Model HM-671 Remote Emergency Call Station is available in a high-impact, white, molded ABS plastic plate. Maximum plate dimensions are 2.8"W X 4.6"H X 0.75"D overall. The switch will not protrude more than 0.5" from finished wall.

INSTALLATION:

Model HM-671 is installed to a single gang electrical box or plaster ring.